FLUID DIAMOND®

REF USA-FDX

FLAGSHIP
Revolutionizing Today's Surgical Suites
SURGICAL

EN Suction Floor Aspirator, suction tubing included

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

PRECAUTIONS

The Fluid Diamond® is a one-time use, disposable product. It cannot be cleaned and should never be reused to help prevent cross-contamination throughout the operating room.

INSTRUCTIONS FOR USE

- 1) Remove the Fluid Diamond® and included suction tubing from the plastic packaging.
- 2) Attach one end of the suction tubing to the Fluid Diamond® suction port and the other end of the suction tubing to a fluid collection canister or other suction source.
- 3) Place the Fluid Diamond® on the floor where fluids build up. Reposition the Fluid Diamond® as needed during use to suction and collect fluids on the floor.
- 4) After use, disconnect the Fluid Diamond® from the fluid collection canister or other suction source and dispose of according to facility regulations and guidelines.

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EN These terms and conditions apply to your ("Customer") purchase and use of the Fluid Diamond® ("product") provided by Flagship Surgical, LLC (the "Company").

WARRANTY. For a period of forty-five (45) days after delivery of the Product to Customer (the "Warranty Period"), the Company warrants that the Product will be free from any material defects when used in accordance with the provided instructions. Customer shall notify the Company in writing of any warranty claim and if the warranty claim is properly made during the Warranty Period, the Company's sole obligation, and Customer's exclusive remedy shall be for the Company to replace the defective Product or, in the Company's sole discretion, provide Customer with a full refund of the purchase price paid for the defective Product.

DISCLAIMER OF WARRANTIES. EXCEPT AS OTHERWISE SET FORTH HEREIN THE PRODUCTS ARE PROVIDED "AS IS" WITHOUT ANY WARRANTIES OF ANY KIND AND THE COMPANY HEREBY DISCLAIMS ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. The limited warranty set forth above gives Customer specific legal rights. Customer may have additional rights that vary

from jurisdiction to jurisdiction. THE COMPANY SHALL NOT BE LIABLE FOR ANY USE OF OR CIRCUMSTANCES RELATING TO CUSTOMER'S IMPROPER USE OF THE MAT, INCLUDING, BUT NOT LIMITED TO, CUSTOMER'S OWN NEGLIGENCE.

LIMITATIONS OF LIABILITY. IN NO EVENT WILL THE COMPANY'S CUMULATIVE LIABILITY EXTEND BEYOND CUSTOMER'S ACTUAL, DIRECT, PROVABLE DAMAGES IN AN AMOUNT NOT TO EXCEED THE AMOUNT PAID BY CUSTOMER FOR THE MAT. IN NO EVENT WILL THE COMPANY BE LIABLE FOR ANY PUNITIVE, AGGRAVATED, EXEMPLARY, SPECIAL, CONSEQUENTIAL, INCIDENTAL OR INDIRECT DAMAGES OR ANY LOSS OF PROFITS, LOSS OF

BUSINESS OR GOODWILL HOWEVER CAUSED, WHETHER FOR BREACH OF WARRANTY, BREACH OF CONTRACT, TORT, NEGLIGENCE, OR OTHERWISE, EVEN IF THE COMPANY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

GOVERNING LAW. The terms hereof shall be governed by and construed in accordance with the laws of the State of New Jersey (USA), without regards to its conflicts of law principles.

CONFLICTS. In the event Customer issues a purchase order in connection with the purchase of the Mat, the terms and conditions contained herein shall control if such purchase order contains terms and conditions that are inconsistent with or conflict with the terms contained herein.

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© Copyright Flagship Surgical®, LLC